



► Foracare Ear Thermometer (IR20b)


In the box you will find:

- Foracare Thermometer
- Thermometer Probe Covers
- 2 AAA Batteries





Getting Started

- 1) Remove the thermometer, probe covers, and batteries from the box.
- 2) Remove the battery compartment cover (A), located below the power button (B), and insert the batteries.
- 3) Replace the cover on the thermometer.

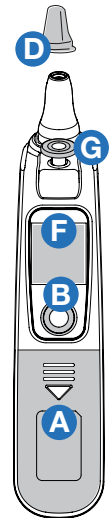

Pairing and Using the Device with the Hub

- 1) Make sure the **Vitals Hub is ON** – connected to power, displaying a **SOLID RED** light (C), and located near the thermometer while taking a reading.
- 2) Gently fit the tip of the thermometer with a clean probe cover (D).
- 3) Press the power button (B) to turn on the thermometer.
The thermometer will beep.
- 4) Place the covered tip of the thermometer into the ear canal.
- 5) Press the button on the back of the thermometer to take a reading (E).
You will hear a beep indicating the measurement is complete and the thermometer can be removed.
- 6) The temperature will display on the screen (F). If the screen is **GREEN** your temperature is normal. If the screen is **RED** your temperature is outside the normal range, which may indicate a fever.
- 7) The **BLUE** light on the Hub will light up and begin BLINKING indicating the reading is being transmitted (G). Once the **BLUE** light turns off, the transmission is complete.
- 8) Press the ejection button to discard the used probe cover.
The thermometer will power off automatically.

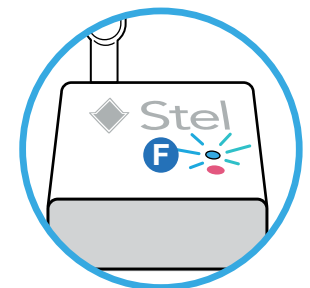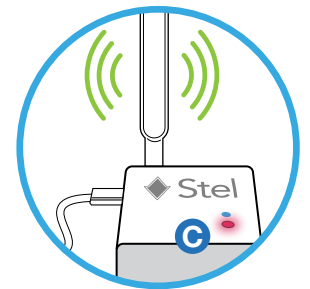

Note: If you have any questions – or if you don't see a blinking blue light after taking a measurement, please contact your care team for assistance.

 *Read the device manufacturer instructions for operation and safety before use.*